



CAUTION

Before use, refer to the CAPR® User's Instructions, P/N 08023040 (received with 2086-03 CAPR IH Helmets) for Base System Details.



Front Lens Mount Side Lens Mount Rear Side Snap Rear T-Tab

REF 2061-11

P/N 01031727

Symbol Definitions



Warning, Caution, or Note



Catalog Number



NIOSH Number



WARNING

Use only if package is received unopened and contents are undamaged. If damage is noted, contact the shipper for replacement or repair.

DO NOT use if any component is damaged. If any components are damaged or contaminated and therefore unfit for safe and effective use, they should be replaced immediately.

Failure to follow User Instructions P/N 08023040 and the instructions contained herein may be hazardous to the user's health.

Only trained and experienced personnel who have read and understand the User Instructions should use MAXAIR Products.

Prior to using any MAXAIR® System or component, be sure to be familiar with the system's NIOSH approved configuration.

The institution using this product in any application is responsible for determining the appropriateness of this equipment relative to regulatory requirements. Bio-Medical Devices Intl, Inc. does not recommend the appropriate systems for a particular institution or facility.

Use only MAXAIR NIOSH approved respirator configurations.

NOT for use in atmospheres immediately dangerous to life or health (IDLH), and atmospheres containing less than 19.5% oxygen or more than 25% oxygen.

Flammability Level I: fabric may burn if exposed to open flame.

Follow current local regulations governing biohazard waste to safely dispose of single use MAXAIR Products.

If you need more information, contact your BMDI Sales Representative, or call BMDI customer service at 1-800-443-3842.

Intended Use

The 2061-11 Filter Cover Cap (FCC) is used with MAXAIR CAPR Helmet Cuff and Shroud outfits that require a MAXAIR Filter. It provides cover for the filter and attachment points for the Cuff and Shroud Lenses. It is a component of a NIOSH approved PAPR100-N system.

Included with each P/N 2061-11



1. Three 2061-11 FCCs
2. One User Instructions
3. Three sets of adapters for MAXAIR CAPR Helmets without those adapters.

Specifications

MAXAIR Recommended System Temperature Limits:

Use/Handling: 0°C to 54°C at a maximum 80% Relative Humidity.

Charging: 0°C to 45°C at a maximum 80% Relative Humidity.

Storage: 0°C to 35°C at a maximum 80% Relative Humidity.

All MAXAIR Systems, components, and disposables are latex free.

Materials

Polycarbonate Cap and Polypropylene Skirt

Regulatory


NIOSH



WARNING

Never mount a FCC on a Helmet until a fully functional Filter is securely in place on the Helmet.

Assembly

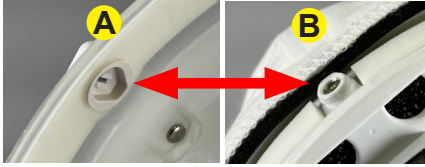

1. Align the inside of the FCC Front Lens Mount (A) to the Helmet Front Adapter (B) and press the Front Adapter into the inside of the Lens Mount (C).




2. Holding the front of the Helmet together with the front of the FCC (A), pull the FCC over and down onto the Helmet (B).



3. Pull the FCC fully down on the Helmet (Squeeze together) to align the FCC T-Tab hole (A) over the Helmet rear Bottom Snap (B).



4. Ensure that the T-Tab is fully snapped down on the Helmet rear Bottom Snap (use thumb if necessary to press in place).

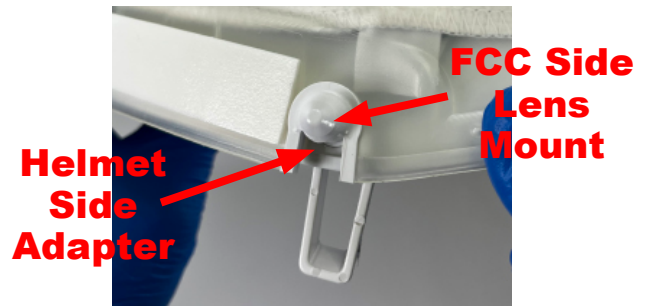

5. Ensure that each FCC Side Lens Mount is fully down over each Helmet Side Adapter.

Disassembly



1. With one hand, hold the Helmet and FCC. With the other hand, pull up and out on the FCC T-Tab and unlatch it from the Helmet Rear Bottom Snap.



2. Continue pulling up on the FCC T-Tab (A) and then simultaneously pull the Helmet down (B), until the FCC is up and off the helmet (C).

