



CAUTION

Before use, thoroughly read MAXAIR® CAPR® P/N 03521015 User Instruction (received with all CAPR Helmets and available at www.maxair-systems.com>SUPPORT>USER INSTRUCTION MANUALS.



O.N. 2099-03
(3sets/box)

Intended Use

2099-03 Liner Pads are exclusively used for MAXAIR CAPR System configurations that include a 2061-04A Hard Hat; a CAPR Liner with these pads may be used on any other CAPR configuration if so desired.



WARNING

2099-03 Pads must be used on all MAXAIR System configurations that include a 2061-04A Hard Hat.

Specifications

Material

Saturated Styrenic Block Copolymer

Recommended System Temperature Limits

Use/Handling: 0°C to 54°C at a maximum 80% Relative Humidity.
Charging: 0°C to 45°C at a maximum 80% Relative Humidity.
Storage: 0°C to 35°C at a maximum 80% Relative Humidity.

Regulatory

NIOSH

Symbol Definitions



Warning, Caution, or Note

O.N.

Order Number

PN

Part Number



WARNING

Failure to follow User Instructions P/N 03521015 and the instructions contained herein may be hazardous to the user's health.

Use only if package is received unopened and contents are undamaged. If damage is noted, contact the shipper for replacement or repair.

Prior to using any MAXAIR® System or component, be sure to be familiar with the system's NIOSH approved configuration.

DO NOT use if any component is damaged. If any components are damaged or contaminated and therefore unfit for safe and effective use, they should be replaced immediately.

Only trained and experienced personnel who have read and understand the User Instructions should use MAXAIR Products.

The institution using this product in any application is responsible for determining the appropriateness of this equipment relative to regulatory requirements. Bio-Medical Devices Intl, Inc. does not recommend the appropriate systems for a particular institution or facility.

Use only MAXAIR Systems/ NIOSH approved compatible components.

NOT for use in atmospheres immediately dangerous to life or health (IDLH), and atmospheres containing less than 19.5% oxygen, or more than 25% oxygen.

Flammability Level I: fabric may burn if exposed to open flame.

Follow current local regulations governing biohazard waste to safely dispose of single use MAXAIR Products.


If you need more information, contact your BMDI Sales Representative, or call BMDI customer service at 1-800-443-3842.



CAUTION

Ensure the correct pad is placed in the correct position on the Liner.

VIEW - Liner from Underside

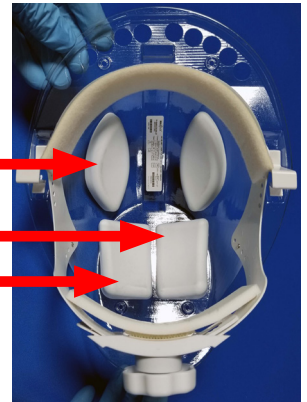


Front Pad
(both sides
are the same)

Right Rear Pad

Left Rear Pad

VIEW - Liner from Top

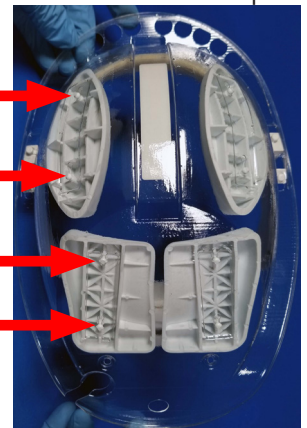

Front Mounting Post

Rear Mounting Post

Front Mounting Post


Rear Mounting Post

Assembly - To Liner



Mounting Posts in fully assembled position.
Mounting post
Mounting slot

Mounting Posts properly assembled to the Liner.

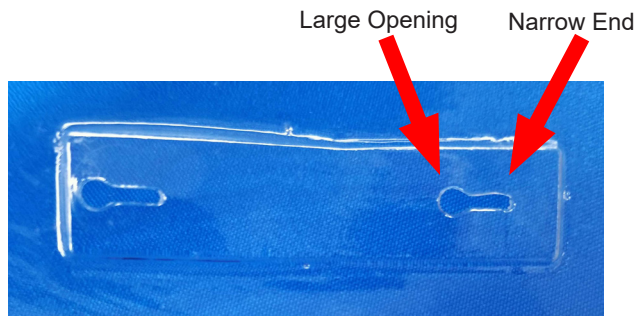

Liner Mounting Slots



1. From the underside of the Helmet Liner, position a Pad Rear Mounting Post under the appropriate Liner Mounting Slot.



2. Press the Pad Rear Mounting Post completely through the large opening in the Liner Mounting Slot.




3. Push the top of the Mounting Post all the way to the back of the narrow end of the Mounting Slot.




4. Ensure that the Mounting Post is all the way up, into, and to the rear of the narrow end of the Mounting Slot.

5. Repeat 1-4. for each Pad that needs to be assembled to the Liner.

Disassembly - From Liner

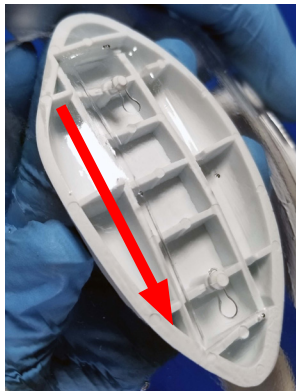

1. Grasp the pad by its sides and push forward (towards front of the Liner) to move the Mounting posts towards the large opening end of the Mounting Slots.

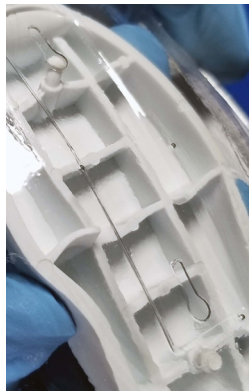

2. Continue to push the Pad forward while gently pulling it down from the Liner until the Mounting Posts are in the Mounting Slots large openings and the Pad can be pulled free from the Liner.

3. Repeat 1.-2. to remove all Pads that need replacement.